

PACKAGE INSERT FOR PERSONAL USE KIT

Read all package insert directions carefully before sample collection.
Test results may be invalid if test is not performed properly.

Sample Deposit


1. Place supplied collection paper inside toilet bowl on top of water.
2. Deposit stool sample on top of collection paper.
3. Collect sample from stool before paper sinks and stool sample touches water.
4. Flush. Collection paper is biodegradable and will not harm septic systems.

Sample Collection

1. Fill in all required information on the sampling bottle.
Open green cap by twisting and lifting




2. Scrape the surface of the fecal sample with the sampling probe.
Cover the grooved portion of the sampling probe completely
with stool sample.

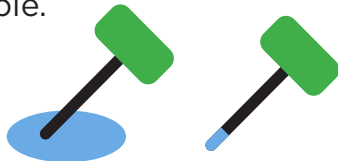

3. Close sampling bottle by inserting the sampling probe
and snap green cap on tightly. Do not reopen.

**Promptly return the sampling bottle to your doctor or laboratory in
envelope provided. A delay may make the test invalid.**

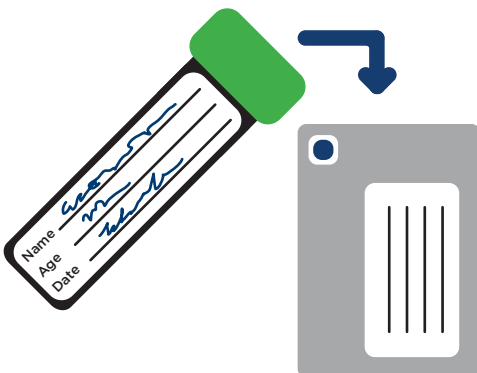

Instrucciones para el Kit de Uso Personal

Lea cuidadosamente las instrucciones antes de recolectar la muestra. Los resultados pueden ser inválidos si la prueba no se realiza correctamente.

Depósito de la muestra


1. Ponga la hoja de recolección en la taza del escusado, por encima del agua.
2. Deposite la muestra de excremento sobre la hoja de recolección.
3. Tome la muestra de excremento antes de que la hoja de recolección se hunda y la muestra de excremento toque el agua.
4. Baje el agua del escusado. La hoja de recolección es biodegradable y no afectará el drenaje

Sample Collection

1. Anote la fecha y hora en la línea de la fecha, y toda la información requerida en la botella de la muestra. Para abrir la botella, gire y jale la tapa verde.



2. Pase el pequeño tubo de recolección por la superficie de la muestra fecal. Cubra toda la parte rayada del tubo de recolección con la muestra de excremento.

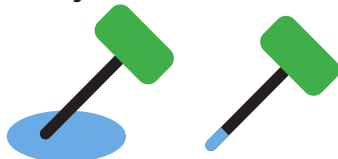

3. Para cerrar la botella de muestra, inserte el tubo de recolección y cierre bien la tapa verde. No vuelva a abrir la botella.

Devuelva la botella de muestreo a su médico o al laboratorio en el sobre proporcionado. Por favor, NO envíe por correo a polymedco.

